1


At complete deployment the NCompass stone extractor features standard openings at the proximal portion of the basket to allow stone entry.

PROCEDURE


2

The multi-wire area at the distal portion of the basket captures the stones as the basket is withdrawn through the ureter preventing stone migration.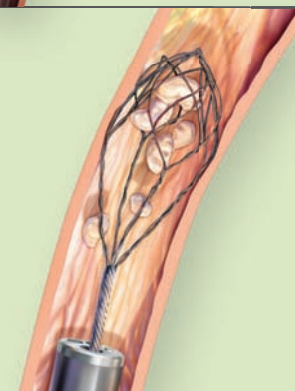

3

As the NCompass stone extractor is gradually closed, the intricate multi-wire geometry completely encompasses the stones for secure capture and extraction.

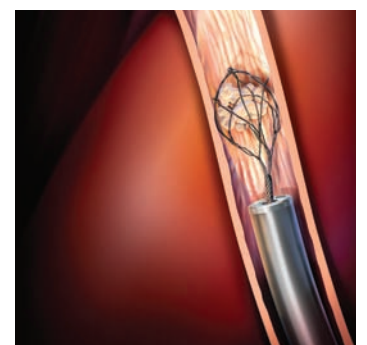
Cook introduced the world to NCircle Nitinol Tipless technology in 1998, a revolutionary advance in the area of stone manipulation. The NCompass Nitinol Stone Extractor demonstrates our continued commitment to being the world leader in nitinol stone extractor technologies.

Global Product Number	Order Number	French Size	Length cm	Basket Diameter cm	Multi-Wire Configuration	Remarks
G46027	NCT4-017115	1.7	115	1.0	4 proximal, 16 distal	tipless
G32701	NCT4-024115	2.4	115	1.0	4 proximal, 16 distal	tipless
G32863	NCT4-024115-MB	2.4	115	1.5	4 proximal, 16 distal	tipless

www.cookmedical.com

Cook Europe Shared Service Centre		
Country	Telephone	Email
Austria	(+43) 1-795-67-121	oeorders@cook.ie
Belgium	French (+32) 27-00-16-33	beorders@cook.ie
Belgium	Flemish (+32) 27-00-16-33	nlorders@cook.ie
Denmark	(+45) 38487607	daorders@cook.ie
Distributors	(+353) 61-239240	sscdistributors@cook.ie
France	(+33) 1-71-23-02-69	frorders@cook.ie
Germany	(+49) 6950-072804	deorders@cook.ie
Italy	(+39) 02-69-68-28-53	itorders@cook.ie
Netherlands	(+31) 202-013-367	nlorders@cook.ie
Norway	(+47) 23-16-2968	noorders@cook.ie
Spain	(+34) 91-2702691	esorders@cook.ie
Switzerland	French (+41) 44-8-009-609	frorders@cook.ie
Switzerland	Italian (+41) 44-8-009-609	itorders@cook.ie
Switzerland	German (+41) 44-8-009-609	deorders@cook.ie
Sweden	(+46) 85-876-94-68	seorders@cook.ie
UK	(+44) 20-7365-4183	ukorders@cook.ie
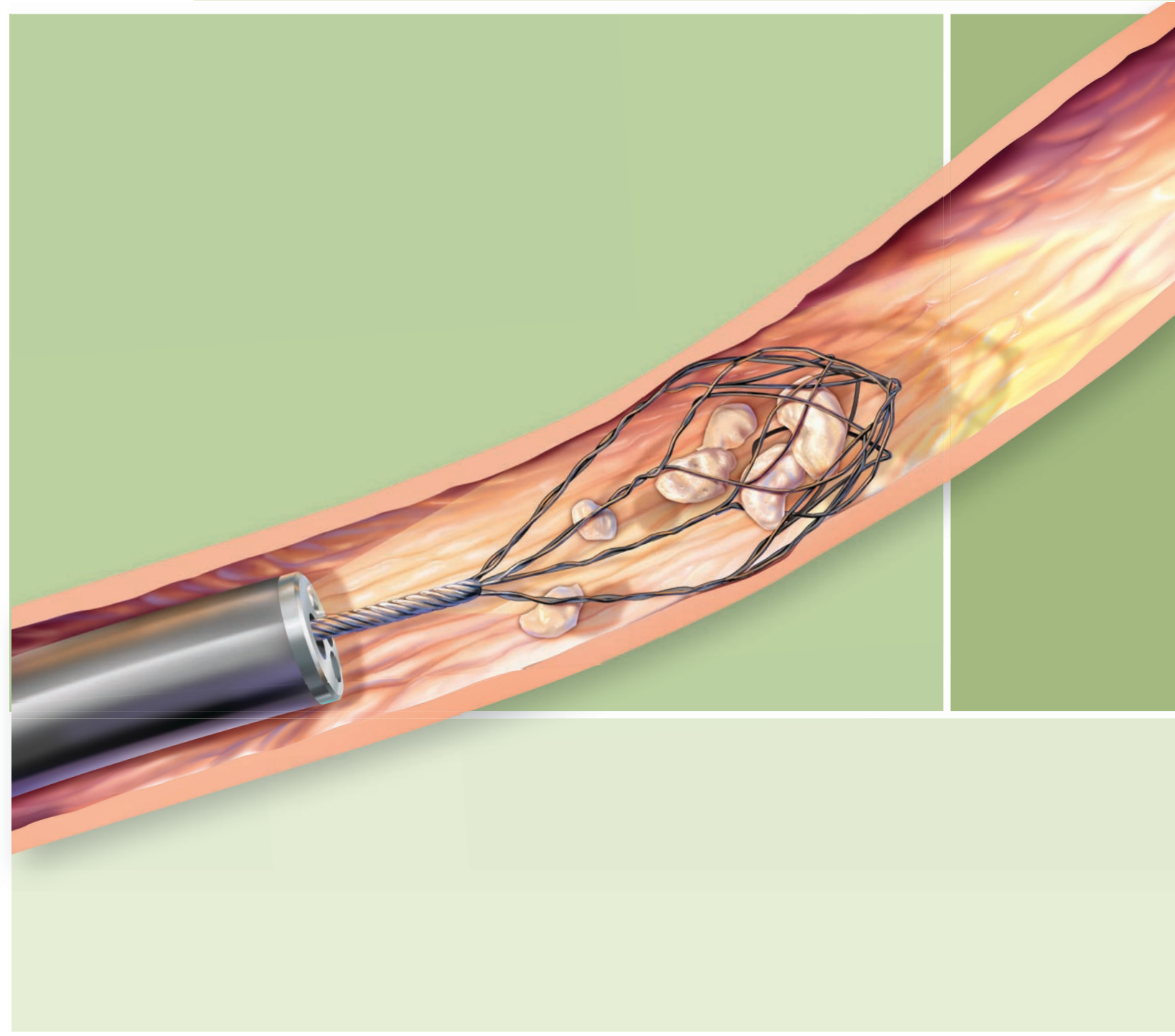
# When stone free is the goal™

The NCompass Nitinol Stone Extractor, featuring multi-wire geometry, is designed to improve stone free rates.

The duality of the NCompass extractor allows both routine capture and extraction of intact stones as well as an improved design to capture multiple small fragments following intracorporeal lithotripsy.

When used in conjunction with the latest minimally invasive stone treatment technologies, the NCompass Nitinol Stone Extractor will assist your efforts in rendering the patient stone free.

## INTRICATE MULTI-WIRE GEOMETRY

Dual basket design; a standard four-wire design at complete deployment, an intricate 16-wire design during closure. At full deployment, distal portion provides a netting effect to help prevent migration of stones.

## TIPLESS DESIGN

The tipless basket design follows in the tradition of the NCircle® basket. Atraumatic design reduces trauma to surrounding tissue.

## NITINOL ALLOY

Resilient memory characteristics retain preformed shape when deployed. Kink resistance creates a durable basket to withstand multiple deployments. Flexibility allows expedient capture and release of stones.

## FLEXIBLE SHEATH

Permits continuous, full deflection of the ureteroscope during stone manipulation.

## 1.7 FRENCH SHEATH

Improves irrigation flow through endoscopic working channel.

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